

# Nasco *Life/form*®

## Intraosseous Blood Mixture

LF01111U Instruction Card



To be used with **Life/form**® Intraosseous Infusion Simulators and Trainers.

### List of Components

- Pint Bottle with Blood Powder
- 4-oz. Tube Lubricating Jelly

### General Instructions for Use

1. To prepare blood mixture:
  - a. Combine 1¼ cups of tap water, the red coloring in the pint bottle provided, and 1 full tube of lubricating jelly. (Measuring cup not included.)
  - b. Shake vigorously for 30 seconds or until contents are mixed completely.

Warning — Blood mixture will stain fabrics, some surfaces, plastics, and skin. Actual product may vary slightly from photo. Nasco reserves the right to change product color, materials, supplies, or function as needed.



ISO 9001 and  
ISO 13485

## Other Available *Life/form*® Simulators

- LF01022U** Fluid Supply Stand
- LF01108U** Infant Intraosseous Infusion Simulator
- LF01109U** Infant Intraosseous Replacement Bones, 10 pack
- LF01130U** 500 ml Fluid Supply Bag
- LF03618U** Child Intraosseous Replacement Bones, 10 pack
- LF03622U** Single Intraosseous Infusion Leg

- LF03632U** Child Intraosseous Infusion/Femoral Leg
- LF03830U** Adult Intraosseous Infusion Simulator
- LF03831U** Adult Intraosseous Bone Replacement Kit, 10 bones and 1 piece of wax
- LF03840U** Adult Articulating Legs for **CRiSis™**, **CPARLENE®**, or "Airway Larry" with Intraosseous Infusion



**LF03830U**



**LF03632U**



**LF03622U**

*Nasco*

901 Janesville Avenue, P.O. Box 901  
Fort Atkinson, Wisconsin 53538-0901

eNasco.com  
1-800-558-9595

E-mail: [lifeform@eNasco.com](mailto:lifeform@eNasco.com)

COPYRIGHT © NASCO 2013 PRINTED IN U.S.A. NP 066-14

